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FOR SUSTAINED LUBRICATION OF THE EYES

Instructions for use

VISMED® GEL MULTI

Sodium hyaluronate from fermentation 0.3%. Transparent hydrogel of natural, highly purified sodium hyaluronate. Devoid of animal proteins. Sterile, free from preservatives.

Composition: 1 ml solution with neutral pH contains sodium hyaluronate 3.0 mg, sodium chloride, potassium chloride, disodium phosphate, sodium citrate, magnesium chloride, calcium chloride and water for injections. The solution is hypotonic.

Indications: Symptoms and signs of dry eye and/or ocular surface damage without restriction in regard to severity, due to diseases such as superficial keratitis, Sjögren syndrome or primary dry eye syndrome. For sustained lubrication of the eyes in case of sensation of dryness, burning and ocular fatigue and other minor complaints of no pathological significance induced, for example, by dust, smoke, dry heat, air conditioning, wind, cold, extended computer screen use, contact lens wear or by ophthalmological screening methods.

Contra-Indications: Individual hypersensitivity to any constituent of the product.

Precautions: Do not touch the tip of the opened container and do not touch the surface of the eye with the tip of the opened container. Put the protective cap back on after using VISMED® GEL MULTI. VISMED® GEL MULTI can be used up to 6 months after first use. Any solution not used within 6 months after opening must be discarded. Otherwise, the sterility is no longer guaranteed and this may be associated with a risk of infection. Do not use VISMED® GEL MULTI if the container is damaged. If discomfort persists while using VISMED® GEL MULTI consult a physician.

Interactions: Do not use VISMED® GEL MULTI at the same time as any drug or other product applied to the eye since it may modify their effects. Wait for at least 15 minutes until instillation of other eye products.

Undesirable effects: In very rare cases (less than 1 in 10,000 patients), transient disorders such as irritation of the conjunctiva, foreign body sensation, pain, redness or burning sensation of the eye, or short-term blurred vision may occur.

Reporting of side effects: Any serious incident that has occurred in relation to the device should be reported to the manufacturer TRB CHEMEDICA AG and the local competent authority of the user.

Dosage and administration: Tear off the tamper-evident seal before the first administration. Remove the protective cap. Tilt the head backwards and position the tip of the container above the eye to be treated. Move the lower eyelid down with the forefinger of one hand. Press the container and one drop of VISMED® GEL MULTI will be released (see illustrations).

If not otherwise recommended, instill 1 or 2 drops of VISMED® GEL MULTI onto the eye as often as needed. After blinking, the solution will disperse and form a transparent and long lasting coating on the surface of the eye. **VISMED® GEL MULTI may also be used while wearing contact lenses (rigid or soft).**

Characteristics and mode of action: VISMED® GEL MULTI contains sodium hyaluronate, a natural polymer, which is also present in the structures of the human eye. The particular physical characteristics of sodium hyaluronate confer to VISMED® GEL MULTI its viscoelastic and water retaining properties. VISMED® GEL MULTI provides a stable coating on the surface of the eye which is only slowly eliminated by blinking. Therefore VISMED® GEL MULTI combines long lasting relief with maximum comfort. VISMED® GEL MULTI is well tolerated due to its unique composition.

Storage and disposal: Store below 30°C in a dry place, protected from light! Do not freeze! Do not use VISMED® GEL MULTI after the expiry date indicated on the container and the folding box. Keep out of reach of children! Empty as well as unused bottles of VISMED® GEL MULTI can be disposed of with household waste.

Presentation: 10 ml solution in a multidose container.

VISMED® GEL MULTI is a medical device for use by patients.

Last revision date: 2024-09

Legend to Illustrations 1-4:

- 1 **Remove the tamper-evident seal.**
- 2 **Remove the protective cap.**
- 3 **Instill one or two drops onto the eye.**
- 4 **Put the protective cap back on after administration.**

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